

REMINDERS FOR BADGE WEARERS



- 1** The technology is a reminder system for proper hand hygiene, not an individual location tracker.
- 2** Whenever your badge turns **yellow** or **green/yellow**, you are still compliant, but you are being reminded that you **MAY** need to perform hand hygiene. If in doubt, wash your hands.
- 3** The system works with both hand sanitizer & soap dispensers, monitored dispensers are identified by the wifi symbol on the front cover of the dispenser.
- 4** When using Manual dispensers, **FULLY** depress the handle. You will only get credit for a full dispense.
- 5** The dispenser will blink a red or gold light to assure you that you've received credit. The light color on the dispenser does not reflect your compliance - check your badge for your own status.
- 6** The dispenser will only communicate with one badge at a time, so make sure to be within arms reach (36 inches). The badge will glow green when successful communication has occurred.
- 7** You will not be counted as in the patient zone until you have been present in the zone for at least 3 seconds.
- 8** The bed beacons will communicate with multiple badges at once as long as they are in the patient zone.
- 9** The badge should be worn anywhere above the waist level for optimal interaction with the dispensers.
- 10** Your badge is unique to you and should be worn at all times while you are at work. If you lose it, please talk to your manager to be issued a replacement.



Questions? Scan the QR code with a Smart Phone to watch the training video



BADGE COMMUNICATION PATTERNS

COMPLIANT



My badge is green. Why? You used a monitored Ecolab hand hygiene soap or sanitizer dispenser.

Action: You can now enter a patient zone. As long as you remain within the patient zone, your badge will remain green and you have unlimited time to safely interact with the patient.



My badge is green and yellow. Why? You have left the monitored patient zone.

Action: You may re-enter the same patient zone you just left within the allotted time (standard is 5 minutes) to remain compliant. If you do not re-enter the patient zone, wash your hands within that time period to remain compliant. If you do not re-enter the patient zone and do not wash your hands, your badge will turn red.

NOT COMPLIANT



My badge is red. Why? You did not wash your hands before, between or after patient contact.

Action: You must perform hand hygiene at a monitored dispenser to reset the badge to green and become compliant before interacting with a patient.

NO IMPACT



My badge is yellow. Why? You have not entered a monitored patient zone or used a monitored soap/sanitizer dispenser for more than 20 minutes.

Action: If you do not enter another patient zone or use a monitored dispenser, the badge will go to sleep; there is no impact to your compliance.

Remember, when in doubt - wash your hands.